
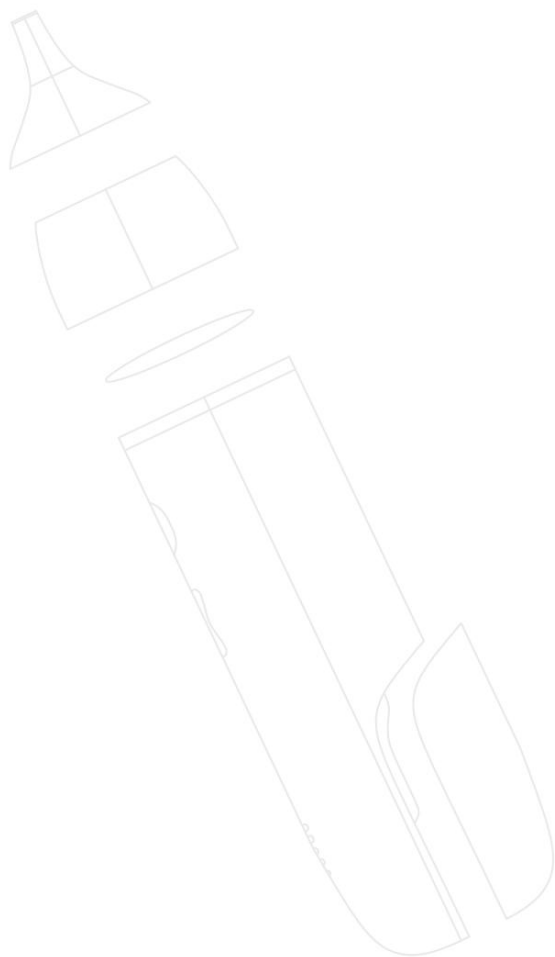
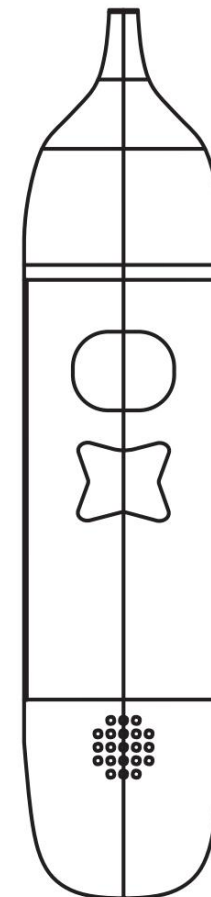
ASPIRADOR NASAL PARA BEBÉ

BABY NASAL ASPIRATOR

MANUAL DE INSTRUCCIONES PARA ARMADO Y USO
INSTRUCTION MANUAL FOR ASSEMBLY AND USE

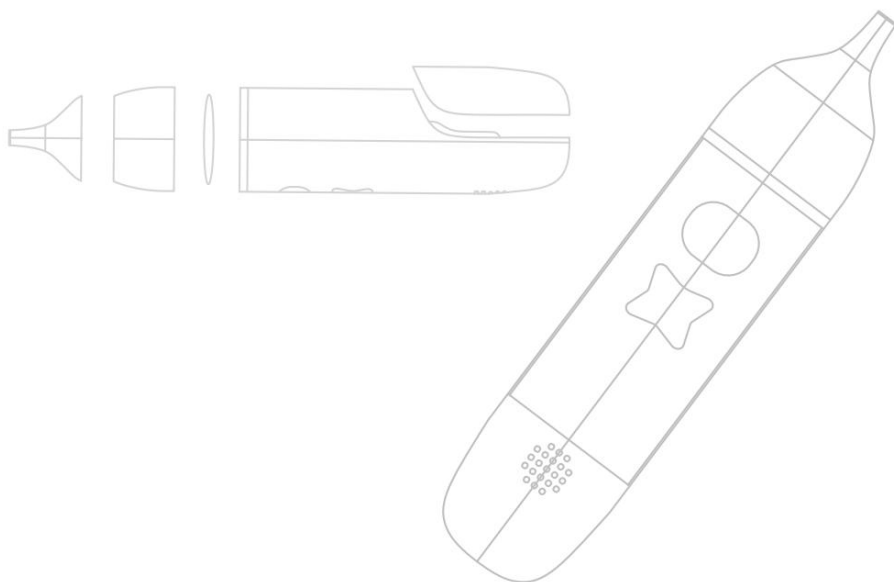belluno[®]
baby 

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Please read this manual in its entirety before assembling the product and follow all instructions contained herein. Please retain this instruction manual for future reference. If the content of this manual does not conform to the actual product, the actual product will prevail. Illustrations are representative only. Design and style may vary. The company reserves design change rights.

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INDEX

ENGLISH

Safety instructions	09
Features	10
Product conservation tips	11
Product instructions	13
Product assembly and cleaning	14
Battery replacement	15

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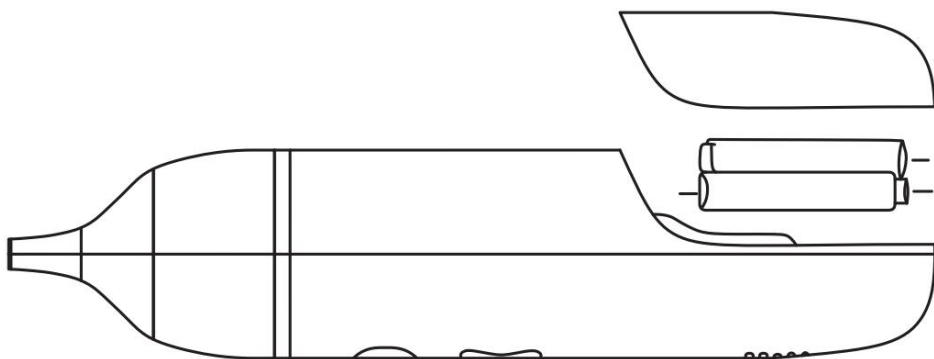
SAFETY INSTRUCTIONS

This product should only be used to clean the nose and derivatives from the baby's nasal cavity.

- Avoid that solid and hard objects of more than 3mm enter the suction nozzle.
- Use the accessories supplied by the manufacturer.
- This product is not a toy. Please store this product in a safe place and do not let the child operate it.
- Please use AA alkaline batteries, or rechargeable batteries, do not use nickel-metal-hydride (NiMH) batteries.
- When not in use for more than two weeks, remove the batteries.
- Please follow the instructions for installing the batteries.

This product is suitable for home use, to help suction the baby's nasal fluid or derivatives from the nasal cavity.

- Please remember to clean the suction nozzle and container before use. It is recommended to soak it in boiling water at 100°C for several minutes. Do not use high temperature devices such as microwave ovens.
- Do not immerse the product completely in liquid.
- Please check that the seal is not damaged.
- Please wash your hands before using the nasal aspirator.
- The operation mode of this product is continuous. To prolong battery life, it is recommended to use it for 10 minutes and then turn it off for 5 minutes.

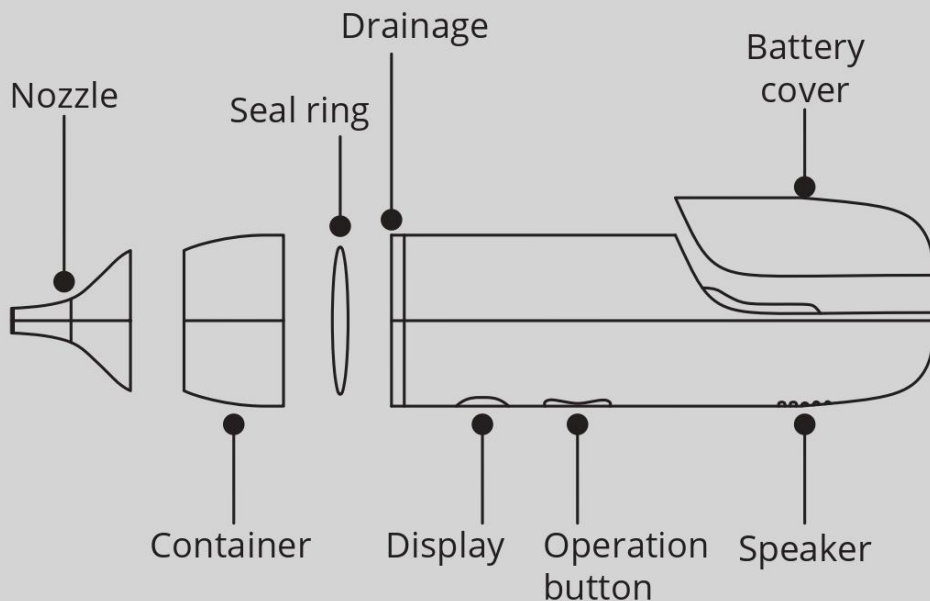
FEATURES

- Three speed suction.
- Beautiful music that makes the baby happy.
- Reusable, easy to use, safe and hygienic accessories.

PRODUCT CONSERVATION TIPS

- Clean the product with a cleaning agent such as alcohol or detergent.
- You can sterilize the suction nozzle with boiling water or steam, but it cannot be sterilized in a microwave oven.
- After disinfection, wipe the product with a soft, dry cloth.
- Keep the product away from direct sunlight and damp places.
- Avoid using corrosive chemicals to clean the product.

When the product is not used for two days, please remove the batteries to avoid damage.

PRODUCT INSTRUCTIONS

- Insert the batteries according to the polarity indications in the battery compartment.
- Press the button "⏻" to activate the baby nasal aspirator.
- Press the button "💡" to adjust one of the three possible positions.
- Carefully place the suction nozzle on the outside edge of the nose and press "S" to clean the nasal cavity.
- After use, first remove the suction nozzle from the outer edge of the nasal cavity, and then press the button "⏻" to turn the device off.
- Before use. You can choose to turn on the music, press "🎵" button to turn on the music, and then press to turn off the music.

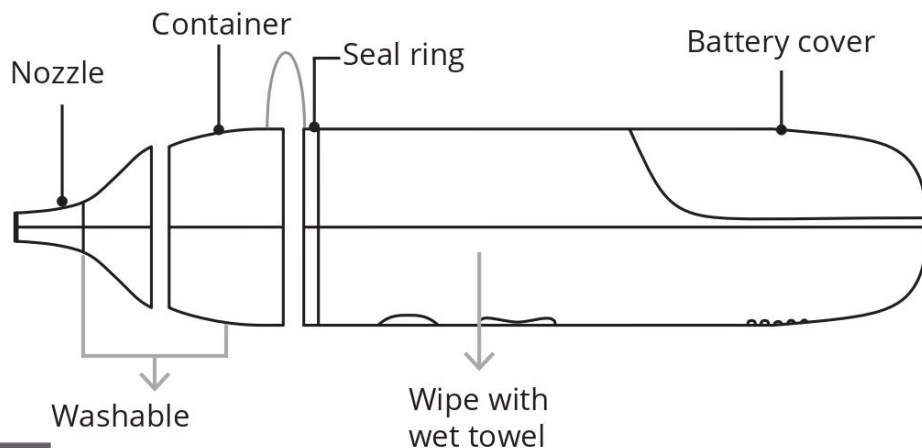
PRODUCT ASSEMBLY AND CLEANING

ASSEMBLY

1. Remove the cover and put in two alkaline batteries.
2. Attach the suction nozzle.

CLEANING

1. The nasal aspirator can only be cleaned with a damp towel.
2. If the aspirator is dirty or has not been used for a week, it can be cleaned with boiling water at 100 °C.

BATTERY REPLACEMENT

1. Remove the battery cover from the baby nasal aspirator.
2. Charge two AA alkaline batteries, making sure the polarity is correct.
3. Replace the battery cover on the baby nasal aspirator.